

Table of contents

\\USER

neuromod

generic

2019-01-22_anat

localizer
acq-0.8mm_T1w
acq-0.8mm_T2w
acq-s3-ap_b0
acq-s3-pa_dwi
dMRI_dir98_AP_test_cmrr
dMRI_dir98_PA_test_cmrr
gre_3d_MTw
gre_3d_PDw
gre_3d_T1w
tfl_b1map_6mm
t1_mp2rage
t2_swi_tra_p2_1.5mm
Localizer
T2w
DWI
GRE-MT1
GRE-MT0
GRE-T1w
GRE-ME

\\USER\neuromod\generic\2019-01-22_anat\localizer

TA: 0:12 PM: ISO Voxel size: 0.5×0.5×7.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HC1-7;NC1,2

Contrast - Common

TR	7.5 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude

Contrast - Dynamic

Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
Base resolution	256
Phase resolution	91 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal

Geometry - AutoAlign

Phase enc. dir.	A >> P
Slice group	2
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.5 ms
Concatenations	3
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	91 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\acq-0.8mm_T1w

TA: 6:38 PM: REF Voxel size: 0.8×0.8×0.8 mmPAT: 2 Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	23.1 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2400.0 ms
TE	2.22 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HC1-7;NC1,2

Contrast - Common

TR	2400.0 ms
TE	2.22 ms
Magn. preparation	Non-sel. IR
T1	1000 ms
Flip angle	8 deg
Fat suppr.	Water excit. fast
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	On
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	23.1 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	2400.0 ms
Multi-slice mode	Single shot
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A29.6 H9.5
R	0.0 mm
A	29.6 mm
H	9.5 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm

System - Miscellaneous

MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

! Position	L0.0 A18.7 H22.3 mm
! Orientation	T > C-20.0
! Rotation	0.00 deg
! A >> P	208 mm
! R >> L	208 mm
! F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2400.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI	1000 ms
Fat suppr.	Water excit. fast
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	7.5 ms
Bandwidth	220 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Performance
Excitation	Non-sel.
RF spoiling	On
Incr. Gradient spoiling	Off
Turbo factor	256

Sequence - Assistant

Mode	Off
------	-----

\\USER\neuromod\generic\2019-01-22_anat\acq-0.8mm_T2w

TA: 5:57 PM: REF Voxel size: 0.8x0.8x0.8 mmPAT: 2 Rel. SNR: 1.00 : spc

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
TE	563 ms
Averages	1.0
Concatenations	1
Filter	Prescan Normalize
Coil elements	HC1-7;NC1,2

Contrast - Common

TR	3200 ms
TE	563 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None
Blood suppr.	Off
Restore magn.	Off

Contrast - Dynamic

Averages	1.0
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	On
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	208
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	0.80 mm
TR	3200 ms
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A29.6 H9.5 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 A29.6 H9.5
L	0.0 mm
A	29.6 mm
H	9.5 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	Off
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

! Position	L0.0 A13.5 H20.8 mm
! Orientation	T > C-20.0
! Rotation	0.00 deg
! A >> P	208 mm
! R >> L	208 mm
! F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	3200 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
----------	-----

Inline - Common

Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Flow comp.	No
Echo spacing	3.52 ms
Adiabatic-mode	Off
Bandwidth	744 Hz/Px

Sequence - Part 2

Echo train duration	1102 ms
RF pulse type	Fast
Gradient mode	Performance
Excitation	Non-sel.
Flip angle mode	T2 var
Turbo factor	314

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\USER\neuromod\generic\2019-01-22_anatlacq-s3-ap_b0

TA: 0:26 PM: ISO Voxel size: 2.0×2.0×2.0 mmPAT: 6 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	57
Dist. factor	0 %
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	2200 ms
TE	71.0 ms
Concatenations	1
Filter	None
Coil elements	HC1-6

Contrast - Common

TR	2200 ms
TE	71.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
Base resolution	110
Phase resolution	100 %
Phase partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	2
Ref. lines PE	40
Accel. factor slice	3

Resolution - iPAT

Reference scan mode	EPI/separate
---------------------	--------------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	57
Dist. factor	0 %
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	2200 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	R2.0 A9.5 H21.4
R	2.0 mm
A	9.5 mm
H	21.4 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	-12.0
> S	3.1

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	21 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	21 mm
MSMA	S - C - T

System - Miscellaneous

Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Performance
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	114 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2200 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	5
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	30

Diff - Body

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	1
b-value	0 s/mm ²
b-value	5
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	30

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.54 ms
Bandwidth	2272 Hz/Px

Sequence - Part 2

EPI factor	110
RF pulse type	Low SAR
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses

\\USER\neuromod\generic\2019-01-22_anat\acq-s3-pa_dwi

TA: 7:41 PM: FIX Voxel size: 2.0x2.0x2.0 mmPAT: 6 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	57
Dist. factor	0 %
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	2300 ms
TE	71.0 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	HC1-6

Contrast - Common

TR	2300 ms
TE	71.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	Fat sat.
Fat sat. mode	Strong

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
Base resolution	110
Phase resolution	100 %
Phase partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

Accel. mode	Slice accel.
Accel. factor PE	2

Resolution - iPAT

Ref. lines PE	40
Accel. factor slice	3
Reference scan mode	EPI/separate

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	57
Dist. factor	0 %
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	2300 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	R2.0 A9.5 H21.4
R	2.0 mm
A	9.5 mm
H	21.4 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	-12.0
> S	3.1

Geometry - Saturation

Fat suppr.	Fat sat.
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	21 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H

System - Miscellaneous

Table position	21 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Performance
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R2.0 A9.5 H21.4 mm
Orientation	T > C-12.0 > S3.1
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	114 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2300 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	4
b-value 1	0 s/mm ²
b-value 2	1000 s/mm ²
b-value 3	2000 s/mm ²
b-value 4	3000 s/mm ²
b-value 1	1
b-value 2	1
b-value 3	1
b-value 4	1
Diff. weighted images	On

Diff - Neuro

Trace weighted images	On
ADC maps	On
FA maps	On
Mosaic	On
Tensor	On
Noise level	30

Diff - Body

Diffusion mode	MDDW
Diff. directions	64
Diffusion Scheme	Monopolar
Diff. weightings	4
b-value 1	0 s/mm ²
b-value 2	1000 s/mm ²
b-value 3	2000 s/mm ²
b-value 4	3000 s/mm ²
b-value 1	1
b-value 2	1
b-value 3	1
b-value 4	1
Diff. weighted images	On
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	On
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	30

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.54 ms
Bandwidth	2272 Hz/Px

Sequence - Part 2

EPI factor	110
RF pulse type	Low SAR
Gradient mode	Performance
Excitation	Standard

Sequence - pTX Pulses

\\USER\neuromod\generic\2019-01-22_anat\DMRI_dir98_AP_test_cmrr

TA: 4:04 PM: REF Voxel size: 2.0x2.0x2.0 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	57
Dist. factor	0 %
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	2334 ms
TE	81.60 ms
Multi-band accel. factor	3
Filter	None
Coil elements	HC1-6

Contrast - Common

TR	2334 ms
TE	81.60 ms
MTC	Off
Magn. preparation	None
Flip angle	78 deg
Refocus flip angle	160 deg
Fat suppr.	None
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
Base resolution	110
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	57
Dist. factor	0 %
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	2334 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	R1.8 A6.5 H22.1
R	1.8 mm
A	6.5 mm
H	22.1 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	-19.6
> S	0.0

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares

System - Miscellaneous

Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	114 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2334 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	98
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	On
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	98

Diff - Body

Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.7 ms
Bandwidth	1684 Hz/Px

Sequence - Part 2

EPI factor	110
Gradient mode	Performance
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	3840 us
Refocus pulse duration	7680 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	On
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	DICOM

\\USER\neuromod\generic\2019-01-22_anat\DMRI_dir98_PA_test_cmrr

TA: 4:04 PM: FIX Voxel size: 2.0x2.0x2.0 mmPAT: Off Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	57
Dist. factor	0 %
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	2334 ms
TE	81.60 ms
Multi-band accel. factor	3
Filter	None
Coil elements	HC1-6

Contrast - Common

TR	2334 ms
TE	81.60 ms
MTC	Off
Magn. preparation	None
Flip angle	78 deg
Refocus flip angle	160 deg
Fat suppr.	None
Grad. rev. fat suppr.	Enabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
Base resolution	110
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	57
Dist. factor	0 %
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	2334 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	3

Geometry - AutoAlign

Slice group	1
Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	R1.8 A6.5 H22.1
R	1.8 mm
A	6.5 mm
H	22.1 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	-19.6
> S	0.0

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Enabled
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares

System - Miscellaneous

Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - All

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R1.8 A6.5 H22.1 mm
Orientation	T > C-19.6
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	114 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2334 ms
Multi-band accel. factor	3

Physio - PACE

Resp. control	Off
Multi-band accel. factor	3

Diff - Neuro

Diffusion mode	Free
Diff. directions	98
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	On
Noise level	40

Diff - Body

Diffusion mode	Free
Diff. directions	98

Diff - Body

Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	3000 s/mm ²
b-value 1	1
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.7 ms
Bandwidth	1684 Hz/Px

Sequence - Part 2

EPI factor	110
Gradient mode	Performance
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	3840 us
Refocus pulse duration	7680 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	On
Invert RO/PE polarity	On
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	DICOM

\\USER\neuromod\generic\2019-01-22_anat\gre_3d_MTw

TA: 3:34 PM: ISO Voxel size: 1.5×1.5×1.5 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	28.0 ms
TE	3.30 ms
Averages	1
Concatenations	1
Filter	Raw filter, Distortion Corr.(3D)
Coil elements	HEA;HEP

Contrast - Common

TR	28.0 ms
TE	3.30 ms
MTC	On
Magn. preparation	None
Flip angle	6 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
Base resolution	128
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off

Resolution - Common

Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	28.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	192 mm
A >> P	192 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	28.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	192 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	210 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\gre_3d_PDw

TA: 3:34 PM: ISO Voxel size: 1.5×1.5×1.5 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	28.0 ms
TE	3.30 ms
Averages	1
Concatenations	1
Filter	Raw filter, Distortion Corr.(3D)
Coil elements	HEA;HEP

Contrast - Common

TR	28.0 ms
TE	3.30 ms
MTC	Off
Magn. preparation	None
Flip angle	6 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
Base resolution	128
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off

Resolution - Common

Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	28.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	192 mm
A >> P	192 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	28.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	192 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	210 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\gre_3d_T1w

TA: 2:18 PM: ISO Voxel size: 1.5×1.5×1.5 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	18.0 ms
TE	3.30 ms
Averages	1
Concatenations	1
Filter	Raw filter, Distortion Corr.(3D)
Coil elements	HEA;HEP

Contrast - Common

TR	18.0 ms
TE	3.30 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
Base resolution	128
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off

Resolution - Common

Slice partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
Slice oversampling	0.0 %
Slices per slab	96
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	18.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	192 mm
A >> P	192 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	5.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	18.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	192 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	210 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\lfl_b1map_6mm

TA: 0:20 PM: ISO Voxel size: 6.0×6.0×6.0 mmPAT: Off Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	20
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	6.0 mm
TR	10000.0 ms
TE	1.84 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D)
Coil elements	HEA;HEP

Contrast - Common

TR	10000.0 ms
TE	1.84 ms
Magn. preparation	None
Flip angle	8 deg
Fat suppr.	None
Water suppr.	None

Contrast - Dynamic

Averages	1
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	6.0 mm
Base resolution	32
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
--------------	-----

Resolution - Filter Image

Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	20
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	6.0 mm
TR	10000.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

Sequence - Assistant

Mode	Off
------	-----

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	192 mm
R >> L	192 mm
F >> H	143 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	Off
Dimension	2D
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Interleaved
Echo spacing	4.1 ms
Bandwidth	490 Hz/Px

Sequence - Part 2

RF pulse type	Low SAR
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On
Turbo factor	32

\\USER\neuromod\generic\2019-01-22_anat\t1_mp2rage

TA: 7:26 PM: ISO Voxel size: 1.2×1.2×1.2 mmPAT: 2 Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	1.20 mm
TR	4000.0 ms
TE	1.51 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(3D)
Coil elements	HEA;HEP

Contrast - Common

TR	4000.0 ms
TE	1.51 ms
Magn. preparation	Non-sel. IR
TI 1	700 ms
TI 2	1500 ms
Flip angle 1	7 deg
Flip angle 2	5 deg
Fat suppr.	None
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	230 mm
FoV phase	100.0 %
Slice thickness	1.20 mm
Base resolution	192
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off

Resolution - Common

Slice partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	176
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	1.20 mm
TR	4000.0 ms
Multi-slice mode	Single shot
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
A >> P	230 mm
F >> H	230 mm
R >> L	212 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	4000.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI 1	700 ms
TI 2	1500 ms
Fat suppr.	None
Dark blood	Off
FoV read	230 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	On

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	3.5 ms
Bandwidth	740 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On
Incr. Gradient spoiling	Off
Turbo factor	132

Sequence - Assistant

Mode	Off
------	-----

\\USER\neuromod\generic\2019-01-22_anat\t2_swi_tra_p2_1.5mm

TA: 4:54 PM: REF Voxel size: 0.9×0.9×1.5 mmPAT: 2 Rel. SNR: 1.00 : swi_r

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A14.4 H27.8 mm
Orientation	T > C-1.1
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	10.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	90.6 %
Slice thickness	1.50 mm
TR	27.0 ms
TE	20.00 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HC1-7;NC1,2

Contrast - Common

TR	27.0 ms
TE	20.00 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	On

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	220 mm
FoV phase	90.6 %
Slice thickness	1.50 mm
Base resolution	256
Phase resolution	96 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Interpolation	Off
---------------	-----

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	L0.0 A14.4 H27.8 mm
Orientation	T > C-1.1
Phase enc. dir.	R >> L
Slice oversampling	10.0 %
Slices per slab	80
FoV read	220 mm
FoV phase	90.6 %
Slice thickness	1.50 mm
TR	27.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L0.0 A14.4 H27.8 mm
Orientation	T > C-1.1
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A14.4 H27.8
L	0.0 mm
A	14.4 mm
H	27.8 mm
Initial Rotation	90.00 deg
Initial Orientation	T > C
T > C	-1.1
> S	0.0

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A14.4 H27.8 mm
Orientation	T > C-1.1
Rotation	90.00 deg
R >> L	200 mm
A >> P	220 mm
F >> H	120 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	27.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	220 mm
FoV phase	90.6 %
Phase resolution	96 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	Yes
Multi-slice mode	Interleaved
Bandwidth	120 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	30 s

\\USER\neuromod\generic\2019-01-22_anat\Localizer

TA: 0:19 PM: ISO Voxel size: 1.0×1.0×6.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	5
Dist. factor	300 %
Position	L0.0 P10.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
Slice group	2
Slices	3
Dist. factor	50 %
Position	L0.0 P2.6 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	38 %
FoV read	500 mm
FoV phase	100.0 %
Slice thickness	6.0 mm
TR	7.8 ms
TE	3.69 ms
Averages	1
Concatenations	8
Filter	Distortion Corr.(2D), Prescan Normalize
Coil elements	HC1,3-7;NC1,2;SP1

Contrast - Common

TR	7.8 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	500 mm
FoV phase	100.0 %

Resolution - Common

Slice thickness	6.0 mm
Base resolution	256
Phase resolution	80 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	5
Dist. factor	300 %
Position	L0.0 P10.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
Slice group	2
Slices	3
Dist. factor	50 %
Position	L0.0 P2.6 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
FoV read	500 mm
FoV phase	100.0 %
Slice thickness	6.0 mm
TR	7.8 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	8

Geometry - AutoAlign

Slice group	1
Position	L0.0 P10.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
Slice group	2
Position	L0.0 P2.6 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P10.0 H0.0
L	0.0 mm
P	10.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Coronal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
Coil Focus	Flat
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.8 ms
Concatenations	8
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	500 mm
FoV phase	100.0 %
Phase resolution	80 %

Physio - PACE

Resp. control	Off
Concatenations	8

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\T2w

TA: 4:02 PM: ISO Voxel size: 0.8x0.8x0.8 mmPAT: 3 Rel. SNR: 1.00 : spcR

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Position	L1.2 P10.8 H6.0 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
AutoAlign	---
Phase oversampling	80 %
Slice oversampling	12.5 %
Slices per slab	64
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	0.80 mm
TR	1500 ms
TE	120 ms
Averages	1.4
Concatenations	1
Filter	Raw filter, Distortion Corr.(3D), Prescan Normalize
Coil elements	HC5-7;NC1,2;SP1

Contrast - Common

TR	1500 ms
TE	120 ms
MTC	Off
Magn. preparation	None
Flip angle	120 deg
Fat suppr.	None
Blood suppr.	Off
Restore magn.	On

Contrast - Dynamic

Averages	1.4
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	0.80 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Position	L1.2 P10.8 H6.0 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
Slice oversampling	12.5 %
Slices per slab	64
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	0.80 mm
TR	1500 ms
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	L1.2 P10.8 H6.0 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
AutoAlign	---
Initial Position	L1.2 P10.8 H6.0
L	1.2 mm
P	10.8 mm
H	6.0 mm
Initial Rotation	90.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Restore magn.	On
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	6 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	6 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
Coil Focus	Flat
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	4.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
Trigger delay	0 ms
TR	1500 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	256 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	3D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Flow comp.	No
Echo spacing	3.99 ms
Adiabatic-mode	Off
Bandwidth	625 Hz/Px

Sequence - Part 2

Echo train duration	311 ms
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
Flip angle mode	Constant
Turbo factor	100

Sequence - Assistant

Allowed delay	30 s
---------------	------

\\USER\neuromod\generic\2019-01-22_anat\DWI

TA: 2:04 PM: ISO Voxel size: 0.9x0.9x5.0 mmPAT: Off Rel. SNR: 1.00 : ezse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	15
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	86 mm
FoV phase	37.5 %
Slice thickness	5.0 mm
TR	610 ms
TE	60.0 ms
Concatenations	5
Filter	None
Coil elements	HC5-7;NC1,2

Contrast - Common

TR	610 ms
TE	60.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	SPAIR
Fat sat. mode	Strong

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	86 mm
FoV phase	37.5 %
Slice thickness	5.0 mm
Base resolution	96
Phase resolution	100 %
Phase partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

Accel. mode	None
-------------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	Off
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	15
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	86 mm
FoV phase	37.5 %
Slice thickness	5.0 mm
TR	610 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	5

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P0.0 H11.0
L	0.0 mm
P	0.0 mm
H	11.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	SPAIR
Fat sat. mode	Strong
Special sat.	None

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	11 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	11 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	90.00 deg
R >> L	86 mm
A >> P	250 mm
F >> H	75 mm
Reset	Off
Couple to	pTx Volume

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	ZOOMit
pTx Volume	1
Vol. Property	Optimization Vol.
Position	Isocenter
Orientation	Transversal
Rotation	0 deg
A >> P	250 mm
R >> L	86 mm
F >> H	75 mm
Vol. Visibility	On

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	Pulse/Trigger
Average cycle	891 ± 142 ms
Average cycle	No Signal ms
Acquisition window	3600 ms
Trigger pulse	1
Trigger delay	0 ms
TR	610 ms
Concatenations	5
Phases	1

Physio - PACE

Resp. control	Off
Concatenations	5

Diff - Neuro

Diffusion mode	MDDW
Diff. directions	30
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	800 s/mm ²
b-value 1	5
b-value 2	1
Diff. weighted images	On

Diff - Neuro

Trace weighted images	Off
ADC maps	Off
FA maps	Off
Mosaic	On
Tensor	Off
Noise level	8

Diff - Body

Diffusion mode	MDDW
Diff. directions	30
Diffusion Scheme	Monopolar
Diff. weightings	2
b-value 1	0 s/mm ²
b-value 2	800 s/mm ²
b-value 1	5
b-value 2	1
Diff. weighted images	On
Trace weighted images	Off
ADC maps	Off
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	8

Diff - Composing

Inline Composing	Off
Distortion Corr.	Off

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.94 ms
Bandwidth	1270 Hz/Px

Sequence - Part 2

EPI factor	36
RF pulse type	Normal
Gradient mode	Performance
Excitation	ZOOMit

Sequence - pTX Pulses

pTX Pulse	1
TX acceleration	1.0

\\USER\neuromod\generic\2019-01-22_anat\GRE-MT1

TA: 2:12 PM: ISO Voxel size: 0.9×0.9×5.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	35.0 ms
TE	3.13 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D)
Coil elements	HC5-7;NC1,2

Contrast - Common

TR	35.0 ms
TE	3.13 ms
MTC	On
Magn. preparation	None
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Interpolation	Off
---------------	-----

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	35.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P0.0 H11.0
L	0.0 mm
P	0.0 mm
H	11.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
-------------------	-----

Geometry - Tim Planning Suite

Table position	H
Table position	11 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	11 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard Neck
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	58 mm
R >> L	230 mm
F >> H	110 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	35.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	230 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	260 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\GRE-MT0

TA: 2:12 PM: ISO Voxel size: 0.9×0.9×5.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	35.0 ms
TE	3.13 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D)
Coil elements	HC5-7;NC1,2

Contrast - Common

TR	35.0 ms
TE	3.13 ms
MTC	Off
Magn. preparation	None
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Interpolation	Off
---------------	-----

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	35.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P0.0 H11.0
L	0.0 mm
P	0.0 mm
H	11.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
-------------------	-----

Geometry - Tim Planning Suite

Table position	H
Table position	11 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	11 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard Neck
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	58 mm
R >> L	230 mm
F >> H	110 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	35.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	230 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	260 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\GRE-T1w

TA: 0:57 PM: ISO Voxel size: 0.9×0.9×5.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	15.0 ms
TE	3.13 ms
Averages	1
Concatenations	1
Filter	Distortion Corr.(2D)
Coil elements	HC5-7;NC1,2

Contrast - Common

TR	15.0 ms
TE	3.13 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	1
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Interpolation	Off
---------------	-----

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice oversampling	18.2 %
Slices per slab	22
FoV read	230 mm
FoV phase	100.0 %
Slice thickness	5.00 mm
TR	15.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P0.0 H11.0
L	0.0 mm
P	0.0 mm
H	11.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
-------------------	-----

Geometry - Tim Planning Suite

Table position	H
Table position	11 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	11 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard Neck
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	58 mm
R >> L	230 mm
F >> H	110 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	10.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	15.0 ms
Concatenations	1
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	230 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Bandwidth	260 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Normal
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	0 s

\\USER\neuromod\generic\2019-01-22_anat\GRE-ME

TA: 4:45 PM: ISO Voxel size: 0.5x0.5x5.0 mmPAT: 2 Rel. SNR: 1.00 : me_r

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	15
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	600.0 ms
TE	14.0 ms
Averages	2
Concatenations	1
Filter	Distortion Corr.(2D), Prescan Normalize, Elliptical filter
Coil elements	HC5-7;NC1,2

Contrast - Common

TR	600.0 ms
TE	14.0 ms
MTC	Off
Flip angle	30 deg
Fat suppr.	None
Water suppr.	None

Contrast - Dynamic

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution - Common

FoV read	224 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
Base resolution	448
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
----------	--------

Resolution - iPAT

Accel. factor PE	2
Ref. lines PE	24
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	On
Mode	2D
Unfiltered images	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	15
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	600.0 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P0.0 H11.0
L	0.0 mm
P	0.0 mm
H	11.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Sat. region	1
Thickness	100 mm
Position	L0.0 A100.0 H0.0 mm
Orientation	Coronal
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	11 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	ISO
Table position	H
Table position	11 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	224 mm
R >> L	224 mm
F >> H	75 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
--------------	----------

System - Tx/Rx

Frequency 1H	123.259601 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	600.0 ms
Concatenations	1

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	On

Inline - Composing

Mode	2D
Unfiltered images	Off

Sequence - Part 1

Introduction	On
Dimension	2D
Combined echoes	3
Flow comp.	Yes
Multi-slice mode	Interleaved
Bandwidth	260 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Normal
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----